



	LATEX	NITRILE	VINYL	CPE
Material	Natural Rubber Latex	Acrylonitrile & Butadiene	Polyvinyl Chloride	Cast Polyethylene
Advantages	Excellent elasticity, dexterity and physical properties	Without protein allergens and excellent chemical resistance	Economical and free from protein allergens	Economical, light weight, without protein allergens or lasticizers
Flexibility	● ● ● ● ●	● ● ● ● ●	● ● ● ● ●	● ● ● ● ●
Fit & Comfort	● ● ● ● ●	● ● ● ● ●	● ● ● ● ●	● ● ● ● ●
Tear Resistance	● ● ● ● ●	● ● ● ● ●	● ● ● ● ●	● ● ● ● ●
Puncture Resistance	● ● ● ● ●	● ● ● ● ●	● ● ● ● ●	● ● ● ● ●
Chemical Resistance	● ● ● ● ●	● ● ● ● ●	● ● ● ● ●	● ● ● ● ●

Standard Specification for Rubber Examination Gloves for Medical Application

Standard Specification for Nitrile Examination Gloves for Medical Application

Standard Specification for Rubber Surgical Gloves

ASTM D3578

ASTM D6319

ASTM D3577



SINGLE USE



NOT MADE WITH NATURAL RUBBER LATEX



STERILIZED USING IRRADIATION



POWDERED



POWDER FREE



FOOD GRADE



EN374 CHEMICAL RESISTANCE



SMOOTH



FULL PALM TEXTURED



FINGER TEXTURED



HACCP CERTIFIED

DO & DON'T

✓ DO

DO wear gloves to reduce the risk of contamination or exposure to blood, other body fluids, hazardous materials, and transmission of infection.

DO clean hands before putting on gloves for a sterile procedure (e.g., insertion of catheter or other invasive device).

DO clean hands after removing gloves.

DO clean hands and change gloves between each task (e.g., after contact with a contaminated surface or environment).

DO make sure that gloves fit you properly before performing any tasks.

DO ensure the correct type of glove is available if you have skin sensitivity or allergy issues.

DO wear gloves in hemodialysis settings for any contact with the patient or the patient's equipment.

DO follow your facility's policy on glove use and remember to consult CDC* and WHO* hand hygiene guidance.

X DON'T

DON'T re-use or wash gloves (except for utility gloves after being properly cleaned).

DON'T substitute glove use for hand hygiene.

DON'T use non-approved hand lotions.

DON'T use gloves if they are damaged or visibly soiled.

DON'T touch your face when wearing gloves.

DON'T wear the same pair of gloves from one patient to another.

DON'T wear gloves in the hall; consult your facility's policy for exceptions.

DON'T forget to remove and dispose of gloves properly.